


Sayana Press Injection Job Aid

Medroxyprogesterone acetate—104 mg/0.65 ml suspension in the Uniject™ injection system

STEP 1: Select an injection site

Sayana® Press can be given on the back of the upper arm, the abdomen (not at the navel), or the front of the thigh. Clean the site if needed.

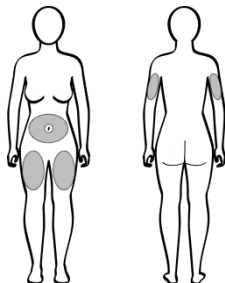


STEP 2: Open the foil pouch and remove Uniject

Check the expiration date.




STEP 3: The Uniject injection system

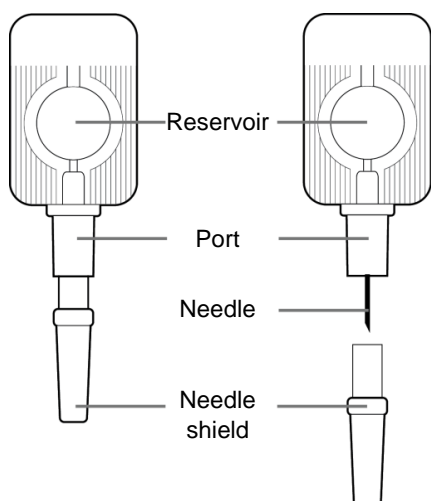


STEP 4: Mix the solution

- Hold the Uniject by the port and shake vigorously for approximately 30 seconds.
- Do not bend the Uniject.
- Check to make sure Sayana Press is mixed and there is no damage or leaking.
- Mix again if there is a delay before you give the injection.

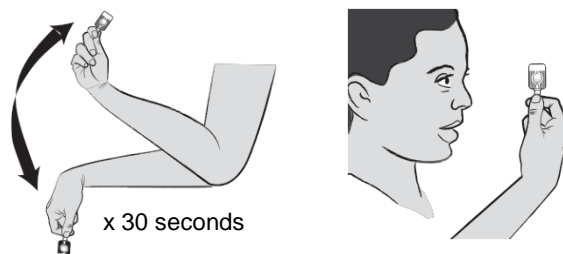

STEP 5: Activate the Uniject

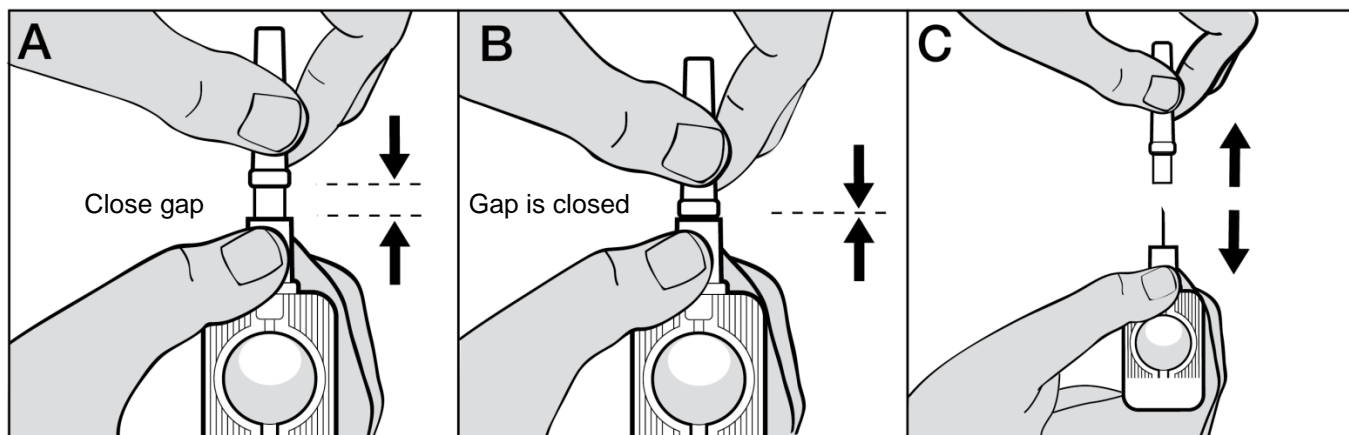

- Hold the Uniject by the port.
- Point the needle upward during activation to prevent dripping.
- Push the needle shield firmly into the port.
- If the gap is not fully closed, you will not be able to squeeze the reservoir during injection.
- Remove the needle shield.


STEP 6: Gently pinch the skin at the injection site

This creates a “tent” for inserting the needle.




STEP 7: Insert the needle at a downward angle

- Continue to hold the Uniject by the port and insert the needle straight into the skin at a downward angle.
- The port should have full contact with the skin to ensure the needle is inserted at the correct depth.

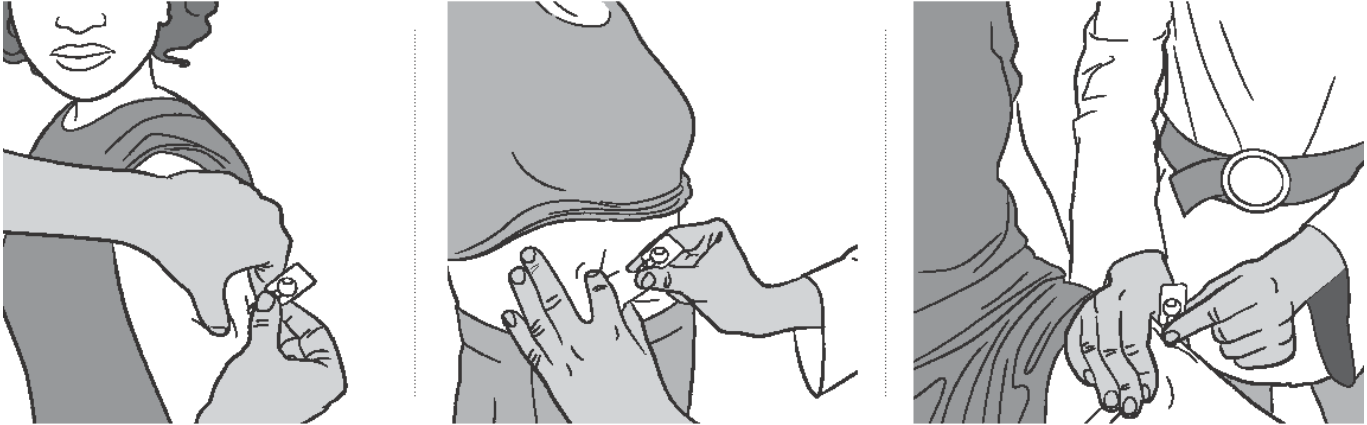

STEP 8: Squeeze the reservoir

- You should not aspirate.
- Squeeze the reservoir slowly (5 to 7 seconds).
- It is OK if there is a little medication left in the reservoir.

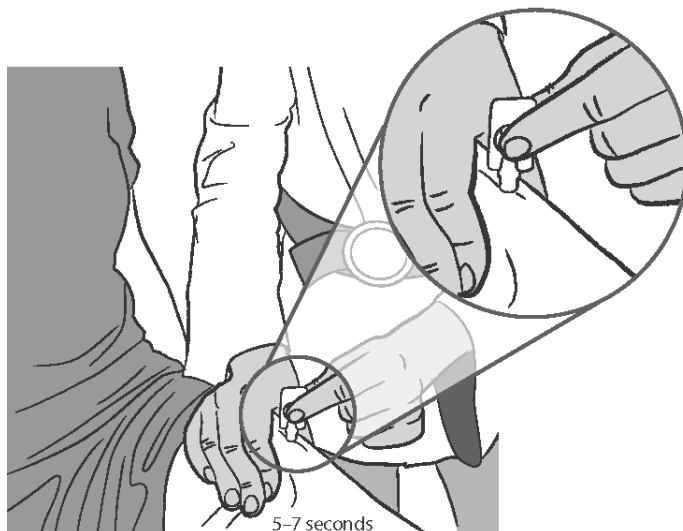

STEP 9: Discard the Uniject

- Do not replace the needle shield.
- Immediately discard the Uniject in a puncture-proof container.

